

# Kodak Ektavision G

## Dental Film

Kodak Ektavision G dental film is a high-resolution extraoral film that captures crisp radiographic detail, ideal for panoramic and cephalometric radiographs, TMJ cases, implant procedures, and more. A low-crossover, 400-speed film system that minimizes radiation exposure, it's designed to be an indispensable tool for your dental practice.



Studies suggest that panoramic radiographs taken with films like Kodak Ektavision G dental film may assist in the detection of calcifications in the carotid arteries, a possible predictor of heart attack or stroke.

# Sharp film for sharper diagnoses

## KODAK EKTAVISION G Dental Film

Carestream Health's sharpest extraoral film designed to visualize the maximum amount of radiographic information for diagnosis.

- Can be processed in automatic equipment or manually
- Use current cassettes with Kodak Ektavision or Lanex screens
- Comes in a variety of sizes to support panoramic and cephalometric radiographs

### Exposure Guidelines for EKTAVISION G Dental Film for Adult Patients\*

Kodak Ektavision extraoral imaging screens or Lanex regular screens are compatible with Kodak Ektavision G dental film.

Examination	Image Receptor Distance	Approximate Exposure in Seconds**		
		65 kVp 10 mA	75 kVp 10 mA	90 kVp 10 mA
Ramus of mandible; body of mandible	15 in	1/15	1/20	1/30
Lateral skull	30 in	3/5	3/10	1/12
Posteroanterior skull (granger technic)	30 in	2/3	1/3	1/5
Temporomandibular articulation	10 in	1/10	1/20	1/30
Cephalometric radiography	60 in	2/3	1/3	1/5

\*For children: Reduce adult exposure time about one-third to one-half.

\*\*Based on half-wave rectified units.

### Ordering Information

Size	Description	Code	Films/Pkgs	CAT Number
5 in x 12 in	Ektavision G	EVG	50	147 7405
15 cm x 30 cm	Ektavision G	EVG	50	837 2716
8 in x 10 in	Ektavision G	EVG	50	185 0809
Cassettes and screens				
15 cm x 30 cm	Ektavision		1 cassette	165 5349
8 in x 10 in	Ektavision		1 cassette	194 4297
Replacement screens for cephalometric and TMJ applications				
8 in x 10 in with foam	Ektavision		1 pair	899 7322
8 in x 10 in without foam	Ektavision		1 pair	148 5390
Replacement screens for panoramic applications				
5 in x 12 in (with double-backed tape)	Ektavision		1 pair	153 9014
4.88 in x 14.75 in (hinged)	Ektavision		1 pair	179 1177
15 cm x 30 cm (with doubled-backed tape)	Ektavision		1 pair	879 3457
15 cm x 30 cm (hinged)	Ektavision		1 pair	152 5674

For more information, call **1-800-933-8031** or visit: [www.kodakdental.com](http://www.kodakdental.com)



Carestream Health, Inc.

©Carestream Health, Inc., 2009.  
Ektavision and Lanex are trademarks of Carestream Health.  
The Kodak trademark and trade dress are used under license from Kodak.