


Using Protective Sheaths

To use a protective sheath, follow these steps:

- 1 Wipe the camera window with a moist lint-free cloth or lens tissue.

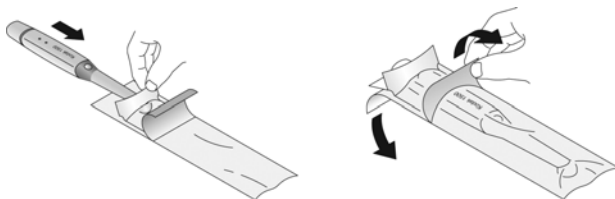

- 2 Place the sheath, paper side down, on a flat surface.
- 3 With the camera window facing down, insert the camera all the way into the sheath between the sheath's white tab and paper backing.
- 4 Hold the sheath in place on the camera by gently pressing the white tab against the camera with your thumb or finger.

- 5 Pull the blue tab on the front of the sheath straight down away from the camera until the paper backing pulls away. Discard the backing, leaving the clear, plastic sheath on the camera.
- 6 Adjust the sheath, if necessary. The end should be tight to prevent flogging or blurring of the image.



Caution: Use a new sheath with each patient and discard the sheath after each use.